Mayborn (UK) Limited,
Balliol Business Park,
Newcastle upon Tyne,
NE12 8EW, UK

Copyright © Mayborn (UK) Limited 2020.
Copyright © Mayborn USA Inc 2020

TOMMEE TIPPEE is a registered trademark.
Mayborn (UK) Limited and Mayborn USA Inc
retain all intellectual property rights to this product

Mayborn USA Inc,
1010 Washington Blvd, 11th Floor
Stamford, CT 06901

Mayborn ANZ Pty Ltd. (ABN 66 154 703 134),
PO Box 475 Mt Waverley VIC 3149, AUSTRALIA
Newborn Brands (NZ) Ltd. PO Box 47-677,
Ponsonby, Auckland, 1144, NEW ZEALAND.

Made in China.
Model number: 423595

Any questions?
Visit our website
tommeetippee.com

travel and microwave steriliser bag

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SL Vrečka za steriliziranje v mikrovalovni pečici in za potovanja

Please read and keep these instructions

423595

13150719_1

EN Travel and microwave steriliser bag

Your Tommee Tippee Travel and Microwave Steriliser Bag is the safe, quick and practical way to sterilise and transport breast pumps and feeding accessories on the go. The bag will hold up to 2 Tommee Tippee Closer to Nature or Advanced Anti-Colic Bottles, or a Tommee Tippee Manual Breast Pump, or 2 Tommee Tippee Silicone Breast Pumps, or a Tommee Tippee Electric Breast Pump (washable parts only). Not for use with a bottle brush or any other metallic items. Once the residual water is removed, your items will be ready for use. Or you can slide the zip seal closed to transport your Items. They will remain sterile for up to 24 hours, as long as the zip remains closed.

See the full range of Tommee Tippee products at tommeetippee.com

Replacement bags can be found at retailers online.

WARNINGS!

Please read instructions before use. Always follow these instructions and check manufacturer’s guidelines to ensure items are suitable for steam sterilisation. Adult use only. This is not a toy. Plastic bags can be dangerous. To avoid the risk of suffocation, keep out of the reach of babies and children. Inspect microwave sterilising bag before each use. Discard immediately at first sign of damage. Only sterilise items which state they are microwave safe. Do not use to sterilise electric breast pump air tubes as they should not be cleaned. Do not over-load. Using less than 60ml (2oz) of water may lead to distortion of items. Only use in a domestic microwave. Do not place in a conventional oven or grill. Do not use in a flatbed microwave. Do not use in a microwave above 1200W. Do not place the sterilising bag on a metallic grill provided with microwaves. Do not use on grill function in a combination microwave oven. The bag should not touch the top of the microwave when inside. Follow the instructions for your microwave. Be sure to follow the time chart as heating longer than the recommended time may distort the shape of items inside. Close the zip seal up to the stop mark. This makes sure there is a 5cm (2 inches) gap for the steam to escape. If no gap is left for steam to escape (i.e. the bag is fully zipped), then the bag will pop open during sterilisation. If this occurs, the items inside the bag will not be sterile and you should dispose of the bag immediately. Always allow to cool for at least 2 minutes before removing from the microwave. The steriliser bag will contain hot water and steam. To avoid risk of scalding and burns take extra care when handling, and only touch cool area. Wash hands thoroughly before handling sterilised items. Record each use and dispose of the microwave sterilising bag responsibly after 30 uses. Tick the boxes on the bag after each use to ensure the bag is only used a maximum of 30 times, do not use more than 30 times.

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